

Single Shade Restoration

Step 1

Choose shade of the overall tooth (A2).

Step 2

Deliver etch and bonding agent to fully prepared tooth. Refer to the Bond section (page 4).

Step 3

Place the first 0.5-1mm layer of composite. Place remaining layers in 2mm increments, replicating the natural anatomy of the surrounding tooth structure. Cure 20 seconds between each layer. Refer to the Restore section (page 5).

Step 4

Shape, finish, polish, and seal. Refer to the Finish section (page 6).



Two Shade Restoration

Step 1

Choose the dentin shade by matching exposed dentin or by matching it to the cervical 1/3 of the tooth.

Step 2

Choose the enamel shade by matching adjacent enamel of the tooth.

Step 3

Deliver etch and bonding agent to fully prepared tooth. Refer to the Bond section (page 4).

Step 4

Place the first 0.5-1mm layer of dentin composite. Continue building up additional layers in 2mm increments, replicating the natural anatomy of the surrounding dentin tooth structure. Cure 20 seconds between each layer. Refer to the Restore section (page 5).

Step 5

Place enamel composite layer building up in 1-2mm increments and replicate the natural anatomy of the surrounding enamel tooth structure. Cure 20 seconds between each layer. Refer to the Restore section (page 5).

Step 6

Shape, finish, polish, and seal. Refer to the Finish section (page 6).

